


PD Tube Identification | Refrigerated

PD Adequacy


Specimens Plasma, 24 Hour Dialysis Fluid, 24 Hour Urine (if applicable)
Container Type(s) 1 Green Top for Plasma, 1 Black Top for Dialysis Fluid, 1 Black Top for Urine (if applicable)

24 Hour Dialysis Fluid (Collection: Batch or Aliquot)
APD/CCPD = Total Overnight + Total Daytime collection
CAPD = Total Drain Volume of each bag collected over 24 Hours

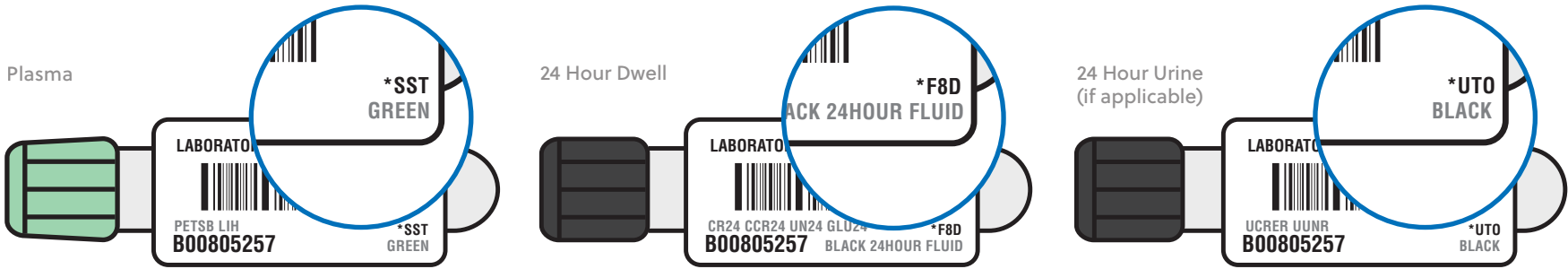


Fluid Overnight Dwell

Specimens Long overnight dwell; 8-12 Hours in length
Container Type(s) 1 Black Top

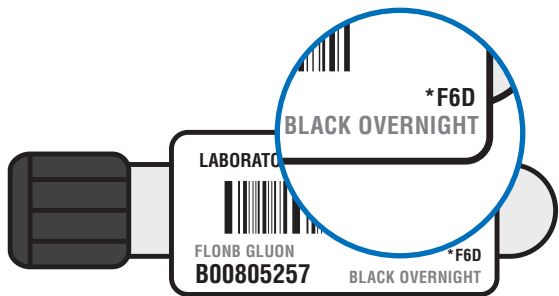

When Ordered Together Two Separate Fluid Specimens Required


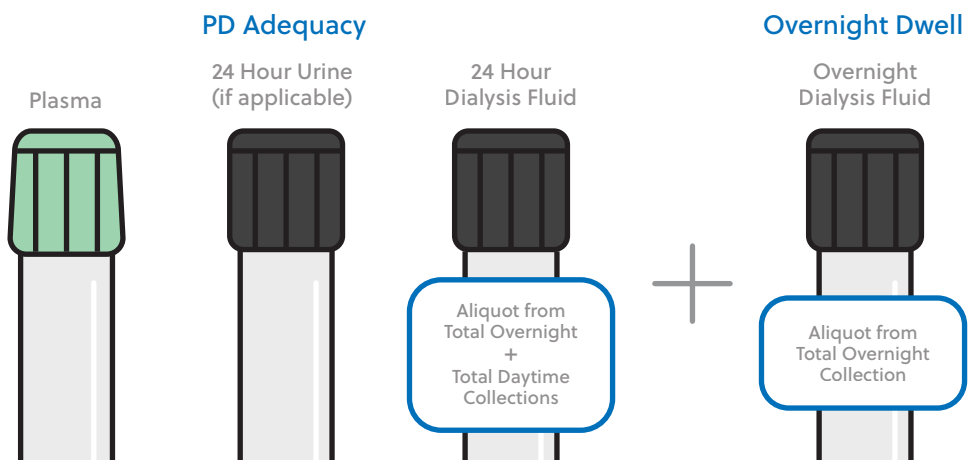


Characterization of Peritoneal Membrane Transport

PET, Standard*

Specimens Plasma, 0, 2, & 4 Hour Dialysis Fluid
Container Type(s) 1 Green Top for Plasma, 3 Black Tops for Dialysis Fluid

*PET, Modified: Same as PET, Standard w/ additional fluid sample drawn at the 1 Hour dwell. Sodium is also measured in all samples. Follow collection instructions in reference manual.

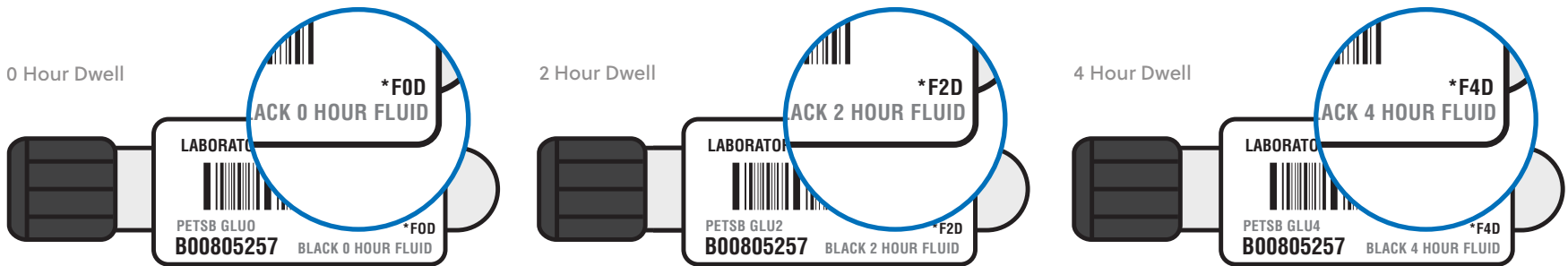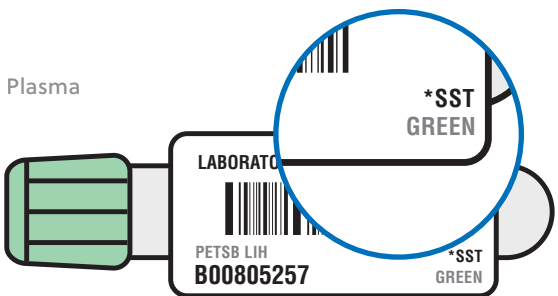

PET, Fast

Specimens Plasma, 4 Hour Dialysis Fluid
Container Type(s) 1 Green Top for Plasma, 1 Black Top for Dialysis Fluid

