


CONFIRM correct patient

CONFIRM correct collection date

SMITH, JOHN
Mn: A12345

Dob: 01/23/1955

07/07/21
Fac: ABC



ALT, BILIT, BUN, CMBP, GLU, LYLES
12345678910

SHORT GREEN

CONFIRM correct tube

Process All Specimens within 2 hours

Follow the
Order of Draw

Immediately
Mix by Inverting

Run Balanced
Centrifuge

Refrigerate



ROYAL BLUE

8-10 times



LIGHT BLUE

4 times



RED

5-10 times


After clotting



SHORT GREEN

8-10 times


Immediately



TALL GREEN

8-10 times


Immediately



LAVENDER

8-10 times



POST



GOLD/WHITE

8-10 times


Immediately

