

# Packaging & Shipping

## Select Proper Box Size & Package by Temperature Type

- ✓ Consolidate all collections by temperature type
- ✓ Must use Specimen Transport Biohazard Bag with Absorbent Sheet
- ✓ Select smallest box possible
- ✓ Use Dual-Temp when shipping both temperature types

| Box Size  | Tube Capacity    |
|-----------|------------------|
| Small     | 1-15             |
| Medium    | 16-100           |
| Large     | 101-200          |
| Dual-Temp | 1-100 per cooler |

### Refrigerated

Ice packs frozen completely flat

Use foam layers

Specimens must not touch ice packs

#### Small & Medium Box


Use 2 ice packs

#### Large Box

Use 4 ice packs



### Room Temperature

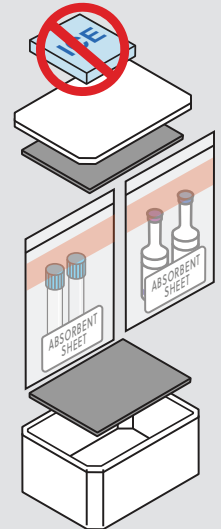

## Shipping Requirements


Ascend packaging meets DOT and IATA shipping and labeling requirements

### Small Box

DO NOT tape sleeve or cooler




Choose **EXEMPT** or **UN3373** Clinical Pak

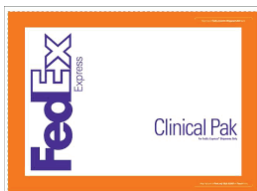

### Medium, Large & Dual-Temp Boxes

Choose **EXEMPT** or **UN3373**

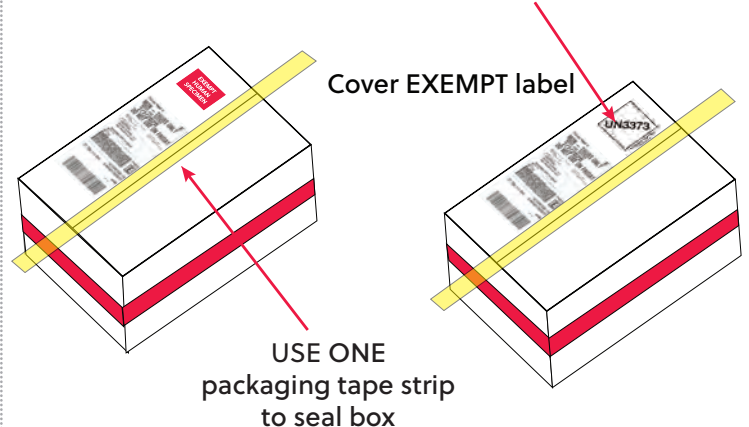

### Box bundles include an envelope containing:

- DOT UN3373 labels required on known or reasonably suspected specimens containing pathogens
- Packaging tape strips
- Mon-Thurs Only shipping labels
- FRIDAY USE ONLY shipping labels required for Saturday delivery