

Peritoneal Dialysis (PD) Tube Identification

PD Adequacy

Specimens Plasma, 24 Hour Dialysis Fluid,
24 Hour Urine (if applicable)

Green Top - Plasma



White Top - 24 Hour Dwell



White Top - 24 Hour Urine (if applicable)



24 Hour Drain Volume Calculation

APD/CCPD = Total Overnight + Total Daytime Collection

CAPD = Total Drain Volume of each bag collected
over 24 Hours

Effluent Culture

Specimens Effluent Fluid

Bactec Culture Bottles



Tall Yellow Top



Fluid Cell Count

Specimens Effluent Fluid

Lavender Top



Peritoneal Equilibration Test (Standard, Fast, Modified)

PET, Standard

Specimens Plasma,
Dialysis Fluid Sample at Hours 0, 2 and 4

White Top - 0 Hour Dwell



Green Top - 2 Hour Plasma



White Top - 2 Hour Dwell



White Top - 4 Hour Dwell



PET, Fast

Specimens Plasma,
Dialysis Fluid Sample at Hour 4

Green Top - Plasma



White Top - 4 Hour Dwell



PET, Modified

Similar to the standard PET, but uses 4.25% glucose solution

Specimens Plasma,
Dialysis Fluid Sample at Hours 0, 1, 2 and 4

White Top - 0 Hour Dwell



White Top - 1 Hour Dwell



White Top - 2 Hour Dwell



White Top - 4 Hour Dwell



Fluid Overnight Dwell

Kinetic modeling software may require a Fluid Overnight Dwell result
in addition to PET, Standard results

Specimens Long overnight dwell; 8-12 Hours in length

White Top - Overnight

